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## Instructions for Use

### 1. Intended use

Herpotherm® is intended for the hyperthermic, local treatment of the symptoms which accompany herpes (Herpes simplex labialis and genitalis) such as burning, tingling, itching or tension.

A herpes outbreak can be prevented by applying the product early. A milder attack or earlier healing can be achieved with a later application.

### 2. Areas of application, indication

Herpotherm® is designed for the symptomatic treatment of herpes in the early stage of a reactive viral infection, i.e. when the first symptoms occur before a blister has formed. The symptoms of the herpes infection can be effectively combated by using Herpotherm. Infection with the Herpes simplex virus cannot, however, be reversed.

### 3. Product components

The Herpotherm® casing surrounded by a removable protective case contains a battery, which is covered by a removable battery cap; a button; a light diode (LED); and a contact patch.

### 4. Instructions for use, first-time use

Treatment with Herpotherm® is reserved exclusively for domestic use by adults. For hygienic reasons and to avoid a transmission of the virus, a Herpotherm® device must only ever be used by one person. Herpes is a viral infection, which is caused by the Herpes simplex virus. Transmission can be caused by droplet infection, through direct contact or by a smear infection.

When you use the device for the first time, remove it from the packaging and insert the enclosed battery. To do this, remove the protective cap and the battery cap and then replace them again once you have inserted the battery.

### Open

### Close

If you know that the device is not going to be used for a long time you should remove the battery, as it could leak and damage the Herpotherm®.

### 5. Mode of operation, product application

Herpotherm® produces a concentrated thermal impulse of an average of 51°C on the contact patch for 4 seconds for the symptomatic treatment of herpes

Herpes viruses are sensitive to temperatures above 50°C. Direct contact with this temperature kills them. This stops the herpes virus from multiplying. Furthermore, the heat shock proteins cause an immediate activation of the immune system, which then quickly stops the herpes blisters from forming.

Herpotherm® should be applied if at all possible when the first symptoms occur.

### 1. Remove the protective case

### 2. Place the contact patch on the affected area and press the button briefly.

The heating-up phase starts when the acoustic signal sounds. The operating temperature has been reached when the LED light comes on; the treatment period of about 4 seconds runs automatically. The treatment ends when the acoustic signal sounds again and the LED light switches off. The Herpotherm® device can now be removed from the area of skin being treated.

**Important: With persistent or recurring symptoms you must wait for a short period of at least 10 seconds before repeating the treatment.**

### 6. Side effects and interactions

No adverse effects are known when the device is used as intended, such as e.g. pathological changes to the skin. Some people may find the concentrated heat uncomfortable. As Herpotherm® is a chemical-free treatment it is also suitable for pregnant women and allergy sufferers.

### 7. General safety instructions

**Important: Herpotherm® must not be used on the eyes.**

### 8. Cleaning, care

The contact patch should be cleaned after every application of the Herpotherm® with a disinfectant solution (e.g. using a Sterillium® disinfectant wipe made by Bode Chemie GmbH, Hamburg). Do not clean the device in water.

### 9. Maintenance, servicing

The medical device is sensitive to environmental influences and should therefore be stored at room temperature, in dry conditions and at low air humidity.

No calibration, care (with the exception of cleaning) or servicing is necessary for this product.

### 10. Troubleshooting

If the product stops working it is probably because you need to change the battery, this is signalled by the LED light blinking three times and the sounding of the acoustic signal.

### 11. Disposal

The Herpotherm® device and spent batteries must not be disposed of in the normal household waste once they have come to the end of their operating life, but should be treated as hazardous waste.

### 12. Product specification, technical data

The medical device complies with the applicable requirements.

Power supply: Ultra CR2 battery 3 V

Visible/audible displays/indicators: the LED light signals the treatment time and if the battery needs changing; an acoustic signal sounds at the start of the heating-up phase and at the end of the treatment time, and also if the battery needs changing.

Emissions: none

Protection class: class I

Climatic class: C (EN 60654-1)

Storage: store at room temperature, in dry conditions and at low air humidity.

Lifespan/Operating life: approximately 100 applications per battery

### 13. Supply specification, packaging

Herpotherm® is packaged with a battery and cleaning wipe in a cardboard box

### 14. Manufacturer / distributor

RIEMSER Arzneimittel AG

An der Wiek 7

17493 Greifswald-Insel Riems

Tel: +49 38351 76-0

Fax: +49 38351 76-48

### Note:

Please follow the instructions for use  
Information publication date: April 2012

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